# by **SIGVARIS**

Gebrauchs- anweisung	2	Návod k použití	28	Kasutusjuhend	52
Instructions for use	4	Upute za upotrebu	30	Інструкція з користування	54
Istruzioni per l'uso	6	Návod na použitie	32	Petunjuk penggunaan	56
Mode d'emploi	8	Navodila za uporabo	34	使用说明	58
Brugsvejledning	10	Uputstvo za upotrebu	36	사용 설명서	60
Bruksanvisning	12	Οδηγίες χρήσης	38	使用方法	62
Käyttöohjeet	14	Kullanım talimatı	40	คำแนะนํการใช้งาน	64
Gebruiks- aanwijzing	16	Használati útmutató	42	გამოყენების ინსტრუქცია	66
Bruksanvisning	18	Инструкции за употреба	44	68 تعليمات الاستخدام	
Instrucciones de uso	20	Инструкции за употреба	44	70 הנחיות שימוש	
Instruções de utilização	22	Instrukțiuni de utilizare	46		
Инструкция по применению	24	Lietošanas instrukcija	48		
Instrukcja użytkowania	26	Naudojimo instrukcija	50		

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DELILAH  
HIGHLIGHT for men  
HIGHLIGHT for women  
HIGHLIGHT MARINIÈRE for men  
HIGHLIGHT MARINIÈRE for women  
MICROFIBER SHADES for men  
MICROFIBER SHADES for women  
SAMSON  
TRAVENO

## Intended purpose

Support stocking to support venous return. The product is intended for multiple use and for application on the legs (lower leg, thigh, entire leg).

## The effects of medical support stockings

The noticeable and controlled compression effect supports the return of blood to the heart, which helps to prevent tired and heavy legs.

## Target group

Support stockings are designed for persons who are engaged in either a standing or sedentary occupation, who travel regularly (car, train, and airplane), and who want to do something to protect their health in a preventative manner.

## Indications & contraindications

### Indications

Prevention of tired and heavy legs during long periods of sitting or standing, even without the presence of varicose veins, prevention of latent swelling, subsequent treatment and prevention following successful varicose vein treatment.

### Contraindications

Medical support stockings must not be worn in the following cases:

- Advanced peripheral arterial occlusive disease (if one of the following parameters applies: ABPI < 0.5, ankle blood pressure < 60 mmHg, toe blood pressure < 30 mmHg or TcPO<sub>2</sub> < 20 mmHg instep).
- Decompensated heart failure (NYHA III + IV)
- Septic thrombophlebitis
- Phlegmasia cerulea dolens

Relative contraindications, whereby benefits and risks must be weighed up, in particular:

- Pronounced exuding dermatoses
- Intolerance to compression material
- Severe sensory disturbances of the extremity
- Advanced peripheral neuropathy (e.g. in diabetes mellitus)
- Primary chronic polyarthritis

## Risks and side effects

Medical support stockings can cause skin necrosis and pressure damage to the peripheral nerves if handled improperly. If problems should arise, the support stockings should no longer be worn.

Itching, scaling, and inflammation may occur under the support stockings in cases of sensitive skin, so adequate skin care under the compression aids is advised.

## Special instructions

Do not cut loose ends on the insides of the stocking.

Protect the support stockings from ointments, oils, and grease to preserve their service life.

Support stockings must not be worn over open wounds.

For products with a grip top: regularly ventilate the skin.

Noncompliance with this information may limit the safety and effectiveness of the product.

All serious incidents which occur when using the product should be reported to the manufacturer and the competent supervisory authority.

## Application

We recommend wearing special gloves to make it easier to put the stockings on and take them off\*. Sharp fingernails and rings can cause defects.

\* **Notice:** Gloves should not be worn for MICROFIBER SHADES for women and for men or for HIGHLIGHT MARINIÈRE for women and for men.

## Donning

1. The best time is after waking up in the morning, when the swelling in your legs has subsided. For easier application, make sure that your legs are dry.
2. Slip the stocking evenly over your heel. Do not pull, but spread the fabric evenly over the leg using the gloves or your hand. While doing so, check for a proper fit at the toe and heel.
3. To protect the seam and the grip top, carefully grasp the knitted fabric and not the seam when donning the support stocking.
4. If it needs repositioning, work the fabric upwards towards the knee with the palm of your hand and spread evenly throughout.

## Correct fit

- Knee high stockings:  
2 finger widths below the back of the knee
- Thigh stockings:  
3 finger widths below the buttocks/crotch

## Doffing

- Take hold of the stockings at the upper edge with gloves or the palm of your hand and pull downwards towards the heel.
- Work the fabric over the heel with the palm of your hand.

For advice on donning and doffing our support stockings, we recommend watching the videos available at [www.sigvaris.com/videos](http://www.sigvaris.com/videos).

### Storage, wearing period, and service life

We recommend wearing the support stockings daily from morning to night. Wearing and washing your support stocking every day may cause the medical pressure gradient and elasticity of your support stocking to diminish.

The shelf life date is indicated on the packaging using the symbol of an hourglass. We guarantee maximum effect for the subsequent six months of wearing.

### Care and storage

Follow the instructions for care on the packaging and/or the textile labels. Wash inside a mesh laundry bag. The stocking must be stored at room temperature and in a dry place away from light. Keep away from direct heat.

### Correct care

Applicable for MICROFIBER SHADES for men / MICROFIBER SHADES for women / TRAVENO

Applicable for HIGHLIGHT MARINIÈRE for men / HIGHLIGHT MARINIÈRE for women / HIGHLIGHT for men / HIGHLIGHT for women / DELILAH / SAMSON

### Material abbreviations:

CO	cotton
EL	elastane
ED	elastodiene
PA	polyamide
PP	polypropylene
PES	polyester
CMD	modal

### Applies to the following products

DELILAH  
 HIGHLIGHT for men  
 HIGHLIGHT for women  
 HIGHLIGHT MARINIÈRE for men  
 HIGHLIGHT MARINIÈRE for women  
 MICROFIBER SHADES for men  
 MICROFIBER SHADES for women  
 SAMSON  
 TRAVENO

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2022 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland
SIGVARIS S.r.l., Via Galileo Galilei 2 / 2, 31057 Silea (TV), **Italy**
SIGVARIS S.A., ul. Powidzka 50, Gutowo Małe, 62-300, Września, **Poland**
SIGVARIS (Australia) Pty. Ltd., 8/53 Stanley Street, West Melbourne VIC 3003, **Australia**
SIGVARIS (SHANGHAI) TRADING CO LTD, Pudong New District, Unit 207, Building 1,