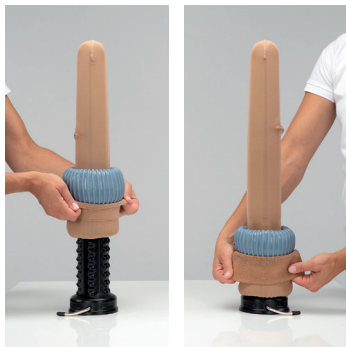
# sigvaris

**DOFF N' DONNER® ROLLY®**  
**DOFF N' DONNER® CONE**

Gebrauchsanweisung	7
Instructions for Use	10
Mode d'emploi	13
Gebruiksaanwijzing	17
Brugsanvisning	21
Bruksanvisning	23
Istruzioni per l'uso	26
Instrukcja użytkowania	30
Petunjuk penggunaan	33
Bruksanvisning	37
Instrucciones de uso	40
Käyttöohje	44
Instruções de utilização	47

5a

5b

6

7

8

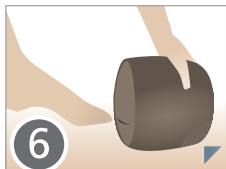
9

10

I

II

III-V

VI

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## English

### Intended purpose

Facilitates donning and doffing of medical compression stockings, and is intended for multiple use.

### Description of function

The DOFF N' DONNER ROLLY is a soft device that allows you to quickly and easily slide medical compression stockings over an arm or a leg. DOFF N' DONNER CONE makes it easier to roll a compression stocking onto a DOFF N' DONNER ROLLY. DOFF N' DONNER CONE also enables a medical compression stocking to be effortlessly unrolled from a DOFF N' DONNER ROLLY.

### Patient target group and intended users

Patients with a thigh circumference between 400 and 600 mm and nursing/care staff.

### Precautions

DO NOT HEAT OR PLACE IN DIRECT SUNLIGHT OR ON HEATED SURFACES! Heat can damage this product. Take care not to drop the DOFF N' DONNER ROLLY as this could result in damage or leakage. Exercise CAUTION when using pointed or sharp objects. The product is NOT toxic.

### Special instructions

Before each use, check the product for signs of wear or deterioration. Optimum functionality of the product can only be ensured if it is completely intact. DOFF N' DONNER ROLLY and

DOFF N' DONNER CONE have only been tested in combination with SIGVARIS GROUP stocking models. SIGVARIS GROUP excludes any and all liability to the extent allowed by law. In particular, SIGVARIS GROUP accepts no liability for improper use of the DOFF N' DONNER ROLLY and DOFF N' DONNER CONE, for user errors or for other harm suffered by the patient as a result of its use, especially when used in conjunction with products not manufactured by SIGVARIS GROUP. Do not affix the DOFF N' DONNER CONE to sensitive surfaces, as this can leave rubber marks. Remove the DND CONE from the surface after use.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

### **Donning using the DOFF N' DONNER CONE**

You can don stockings either with or without using the DOFF N' DONNER CONE. Please follow the instructions below to don stockings using the DOFF N' DONNER CONE.

**1** Place and stick the DOFF N' DONNER CONE on a smooth surface (flip the lever down, DOFF N' DONNER CONE will adhere by suction).

**2** Place the compression stocking with the finished top edge (soft top or grip top) over the DOFF N' DONNER CONE. The inside surface of the stocking rests over the DOFF N' DONNER CONE.

**3** Gently pull the compression stocking completely over the DOFF N' DONNER CONE.

**Closed toe:** Position the toe seam, stretched out smoothly, onto the tip of the DOFF N' DONNER CONE.

**Open toe:** Position the edge of the stocking three finger widths below the tip of the DOFF N' DONNER CONE.

**4** Place the DOFF N' DONNER ROLLY over the DOFF N' DONNER CONE and roll it downward. Pull the finished edge of the compression stocking up over the DOFF N' DONNER ROLLY.

**5a** Calf stocking: Roll the DOFF N' DONNER ROLLY upward until the compression stocking is rolled completely onto the DOFF N' DONNER ROLLY.

**5b** Thigh stocking: Pull the finished edge of the compression stocking carefully beyond the tip of the DOFF N' DONNER CONE, so that the stocking is wrinkle-free. Take hold of the DOFF N' DONNER ROLLY and roll it upward until the compression stocking is rolled completely onto the DOFF N' DONNER ROLLY.

**6** Carefully stretch the toe seam, so that it is clearly visible from the outside. Hold the DOFF N' DONNER ROLLY in such a way that the heel of the compression stocking is visible from the outside and is facing the floor.

**7** Place the toe seam on the toes. The heel of the rolled-up compression stocking should be pointing downward.

**Open toe:** Place the finished cuff as shown by the arrow.

**8** Roll the DOFF N' DONNER ROLLY over the heel up onto the leg.

**9** Once the compression stocking is on the leg, roll the DOFF N' DONNER ROLLY back over the heel.

**10** Smooth the compression stocking over the leg with Sigvaris gloves for a wrinkle-free fit.

### **Donning without using the DOFF N' DONNER CONE**

You can don stockings either with or without using the DOFF N' DONNER CONE. Please follow the instructions below to don stockings without using the DOFF N' DONNER CONE.

**1** Pull the compression stocking over the arm and stretch it out well to prevent creases.

**Closed toe:** Position the toe seam, stretched out smoothly, onto the fingertips.

**Open toe:** Position the edge of the stocking three finger widths below the fingertips.

**2** Put your hand through the DOFF N' DONNER ROLLY and roll it up your arm to the upper end of the compression stocking.

**3** Take hold of the end of the compression stocking and pull it over the DOFF N' DONNER ROLLY. Then roll back the DOFF N' DONNER ROLLY and push the compression stocking into it.

**4** Roll the DOFF N' DONNER ROLLY down to your hand to completely roll up the compression stocking. Afterwards, remove the DOFF N' DONNER ROLLY from your arm.

**5** Carefully stretch the toe seam, so that it is clearly visible from the outside. Hold the DOFF N' DONNER ROLLY in such a way that the heel of the compression stocking is visible from the outside and is facing the floor.

**6** Place the toe seam on the toes. The heel of the rolled-up compression stocking should be pointing downward.

**7** Roll the DOFF N' DONNER ROLLY over the heel up onto the leg.

**8** Once the compression stocking is on the leg, roll the DOFF N' DONNER ROLLY back over the heel. Use the ribbed surface of the DOFF N' DONNER ROLLY to spread the stocking out without creases. We recommend you use Sigvaris gloves to do this.

### Doffing

**a b c** The DOFF N' DONNER ROLLY can be used as illustrated in reverse to aid doffing.

### DOFF N' DONNER PRODUCTS comprise:

**A** DOFF N' DONNER CONE

**B** DOFF N' DONNER ROLLY

The products are available both separately and as a set.

### Indications

- Impaired ability to don and doff compression stockings in people with compromised arm and/or hand and/or finger function and

reduced strength (e.g. those with joint diseases, neurological disorders, injuries, deformities)

- To don and doff compression stockings without assistance

### Contraindications

- There are no contraindications.

### Suitable for the following models:

- A-D knee-high stocking: open/closed toe, soft top or grip top
- A-G thigh stocking: open / closed toe, soft top or grip top

### Storage and service life

**CAUTION:** If used frequently for an extended period of time, the DOFF N' DONNER ROLLY may become brittle and, in exceptional cases, may wear out. We therefore recommend storing the DOFF N' DONNER ROLLY protected from light and heat.

If the product is used and stored properly, we guarantee it will function as intended for 6 months from the date of purchase.

### Cleaning and disinfection

After use, we recommend that the DOFF N' DONNER ROLLY and DOFF N' DONNER CONE be cleaned and/or disinfected. The product is not suitable for machine cleaning. For personal use, we recommend cleaning the DOFF N' DONNER ROLLY in a commercially available plastic bag by simply swirling it around in soapy water. The DOFF N' DONNER CONE should be cleaned with soapy water as required and dried with a clean, lint-free cloth. For institutional use (e.g. in hospitals, doctor's practices, and in medical suppliers), disinfection is mandatory.

The fundamental suitability of the products to be effectively cleaned and disinfected has been verified by an independent, state-accredited, recognised test laboratory using Bacillol® 30 sensitive foam (HARTMANN®) following the specified method.

## WARNING!

Any other cleaning agent and disinfectant must

- contain less than 30% alcohol (disinfectants with a higher alcohol content than this can dry out the material, make it brittle and reduce its elasticity);
- be suitable for cleaning and disinfecting the material composition specified in the instructions for use;
- be suitable for cleaning and disinfecting the surface of non-invasive medical devices; and
- not contain any harsh chemicals such as acetone, bleach, or products containing fragrances, as these can damage the material.

**I** Place the Sigvaris sterilization bag over the DOFF N' DONNER CONE.

**II** Place the DOFF N' DONNER ROLLY over the DOFF N' DONNER CONE and roll it downward.

**III** Take the lower ends of the sterilization bag and pull these upward over the DOFF N' DONNER ROLLY.

**IV** Fill the sterilization bag with 150 ml of the disinfectant (DOFF N' DONNER ROLLY should be lightly soaked in the liquid).

**V** Move the sterilization bag up and down for at least 30 seconds so that the disinfectant is distributed well and evenly through the DOFF N' DONNER ROLLY.

**VI** Pull the sterilization bag with the DOFF N' DONNER ROLLY upwards and close the sterilization bag tightly by twisting the bag. Shake the sterilization bag up and down with the DOFF N' DONNER ROLLY for at least 30 seconds. If no DOFF N' DONNER CONE and sterilization bag are available fill a plastic bag with disinfectant. Make sure the disinfectant is distributed well and evenly around the DOFF N' DONNER ROLLY. Shake the sterilization bag up and down with the DOFF N' DONNER ROLLY for at least 60 seconds.

**VII** Dispose of the disinfectant.

**VIII** Take the DOFF N' DONNER ROLLY out of the sterilization bag. Rinse the DOFF N' DONNER ROLLY with clean fresh water and allow it to dry thoroughly.

**IX** Dispose of the sterilization bag (for one-time use only).

**X** Keep the DOFF N' DONNER CONE wet with the disinfectant for at least 60 seconds. Then dry it with a clean, lint-free cloth.

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SIGVARIS, SIGVARIS GROUP, DOFF N' DONNER and ROLLY are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2024 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland
**SIGVARIS GmbH**