sigvaris

ULCER X Light

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2
3

Intended purpose

Two-layer compression treatment on the leg to support venous return and the associated faster wound healing in *ulcus cruris*.

Effects of medical compression stockings

Compression therapy forms the basis for treating venous disorders, lymphedema and lipedema. Carefully controlled pressure on the tissue improves the blood return in the legs and increases lymphatic flow. At the same time, it improves microcirculation and therefore the supply to the tissue. Compression has a number of effects including reducing edema, encouraging wound healing and reducing the symptoms of varicose veins and venous-related skin changes.

Putting on the understocking (see back page)

1. Turn the understocking inside out at the level of the ankle and pull the understocking carefully over the instep and heel like a normal sock. This inversion technique ensures that the wound dressing remains in place.
2. Take hold of the understocking by the cuff and carefully pull it halfway up the lower leg.
3. Distribute the stocking fabric evenly over the lower leg by gradually easing the understocking up the leg with flat hands and stroking movements. This allows you to distribute the knitted fabric evenly over the lower leg with very little effort.

Putting on the Essential CLASSICAL compression stocking (see back page)

4. Turn the compression stocking inside out at the level of the ankle and pull it carefully over the instep and heel.
5. Take hold of the compression stocking by the cuff and carefully pull it halfway up the lower leg.
6. Distribute the knitted fabric evenly over the lower leg by gradually easing the compression stocking up the leg using flat hands and stroking movements.

Taking off the Essential CLASSICAL compression stocking (see back page)

7. To doff the compression stocking, grip the upper edge and pull the band down over the heel and almost to the toe.
8. Using flat hands and stroking movements, ease the compression stocking over the instep and heel, and towards the toe.

Care and storage

Understocking: max. 60 °C.

Wash daily inside a mesh laundry bag. The stockings must be stored at room temperature and in a dry, dark place. Keep away from direct heat and moisture. The shelf life date is indicated on the packaging using the symbol of an hourglass. We guarantee maximum effect for 6 months of wearing.

Important instructions

- Do not wear Specialities ULCER X Light on open wounds. The wounds should be covered with the appropriate wound dressings.
- The understocking has to be worn for 24 hours. Put the overstocking on after getting up and remove it before going to bed.
- Sharp fingernails and sharp-edged rings can quickly cause defects. Therefore, wear Sigvaris gloves when putting on and removing the stockings.
- Do not cut loose ends on the insides of the stockings.
- Protect the compression stocking from ointments, oils, grease and excessive heat to preserve their service life.

Indication

For the treatment of leg ulcers (*ulcus cruris venosum*) subsequent to wound treatment.

Absolute contra-indications

Advanced peripheral arterial occlusive disease, congestive heart failure, septic phlebitis, phlegmasia coerulea dolens.

Relative contra-indications

Pronounced weeping dermatoses, intolerance to compression stocking material, serious sensitivity disorders of the extremities, advanced peripheral neuropathy (e. g. diabetes mellitus), primary chronic polyarthritis.

Risks and side effects

Medical compression stockings can cause skin necrosis and pressure damage to the peripheral nerves if handled improperly. Itching, scaling, and signs of inflammation may occur under the compression aids in cases of sensitive skin, so adequate skin care is advised.

Noncompliance with this information may limit liability.

All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.

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7

8

To watch video
instructions, please visit
our website:
[www.sigvaris.com/
videos](http://www.sigvaris.com/videos)

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SIGVARIS AG (DMCC Branch), Unit No: AG-05-C, Silver (AG) Tower, Jumeirah Lakes Towers,
Dubai – United Arab Emirates
Phone +971 4 558 6265

SIGVARIS INC., 1119 Highway 74 South, Peachtree City, GA 30269, United States
Phone +1 800 322 7744, certified ISO 13485

SIGVARIS CORP., 8423 Chemin Dalton, Montréal, QC H4T 1V5, Canada
Phone +1 514 336 2362

SIGVARIS DO BRASIL, Av. José Benassi, 2323, Distrito Industrial Fazgran, CEP 13213-085, Jundiaí - SP, Brazil
Tel. +55 114525 5700, certified ISO 13485

If you look for further distributors please consult: www.sigvaris.com

SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG,
CH-9014 St. Gallen/Switzerland, in many countries worldwide.
© 2021 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland

SIGVARIS
GROUP

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SIGVARIS GmbH, Dr.-Karl-Lenz-Strasse 35, 87700 Memmingen, Germany
SIGVARIS S.r.l., Via Galileo Galilei 2/2, 31057 Silea (TV), Italy
SIGVARIS SAS, z.l. Sud d'Andrézieux, Rue Barthélémy Thimmonier, 42170 St-Just-St-Rambert, France
SIGVARIS BRITAIN LTD., 1 Imperial Court, Magellan Close, Andover, SP10 5NT, Great Britain
SIGVARIS S.A., ul. Powidzka 50, Gutowo Male, 62-300 Września, Poland
SIGVARIS (Australia) Pty. Ltd., 207 Victoria Street, West Melbourne VIC 3003, Australia