sigvaris

Compreflex® Standard Thigh Compreflex® Contain Thigh

DE	Gebruiksaanweisung	2	LT	Naudojimo instrukcija	41
EN	Instructions for Use	4	UK	Інструкція з користування	43
IT	Istruzioni per l'uso	6	ID	Petunjuk Penggunaan	45
FR	Mode d'emploi	8	CN	使用指南	47
DA	Brugsanvisning	10	KO	사용 설명서	48
FI	Käyttöohje	12	JA	使用方法	50
NL	Gebruiksaanwijzing	14	TH	คำแนะนำในการใช้งาน	52
NO	Bruksanvisning	17	KA	Пайдалану жөнiндегi нұсқаулық	53
ES	Instrucciones de uso	19	HI	उपयोगकर्ता मैनुअल	56
PT	Instruções de utilização	21	MS	Arahan Penggunaan	57
RU	Инструкция по применению	23	60	تعليمات الاستخدام	AR
PL	Instrukcja użytkowania	25	61	הנחיות לשימוש	HE
SK	Návod na použitie	28			
SL	Navodila za uporabo	30			
EL	Οδηγίες χρήσης	32			
TR	Kullanım Talimatı	34			
BG	Инструкции за употреба	36			
RO	Instrucțiuni de utilizare	38			

Deutsch

Produktbeschreibung

Compreflex® Standard Thigh

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Product description

Compreflex® Standard Thigh

The Compreflex® Standard Thigh is an inelastic product that provides precise compression ranges of 21–46 mmHg thanks to the use of the Accutabs. The product is combined with a pair of Cotton Liners to protect the skin. A waist attachment, which fastens the product in place and prevents slipping, is included with the product.

Compreflex® Contain Thigh

The Compreflex Contain is an adjustable inelastic medical compression device, providing compression of 20–50 mmHg to

the thigh. This compression device is paired with a non-compressive liner, which provides protection for the skin.

Intended purpose

These inelastic wraps are compression therapy devices used on the thigh. Compression therapy improves venous and lymphatic return, which reduces venous and lymphatic edema. It is used in the initial decongestion phase of lymphatic edema and pronounced venous edema as an alternative to wrapping with bandages. This medical device is intended for multiple use and is not sterile.

Effect of medical compression

Compression products apply a carefully controlled pressure on tissue and veins. This improves the blood return in the veins and increases the lymphatic flow. At the same time, it improves microcirculation and therefore the supply to the tissue.

Target group

The intended target group includes patients who have been prescribed a compression product for individual compression therapy by the responsible physician or trained health professional. The target group also includes user groups from health care professions who apply the products to patients.

Indications and contraindications

Indications

- Edema caused by chronic venous disease (CVD) due to obstruction or valvular insufficiency: treatment and/or prevention
- CVD with venous edema, skin changes, open and closed ulcers
- Lymphedema
- Lipedema
- Edema due to immobilization
- Non-specific edema or edema after surgery

Contraindications

- Advanced peripheral arterial occlusive disease PAOD (ABPI <0.6, ankle blood pressure <60 mmHg, toe blood pressure <30 mmHg, or TcPO₂ <20 mmHg instep)
- Severe cardiac insufficiency (NYHA III–IV)
- Septic phlebitis
- Phlegmasia coerulea dolens
- Compression of an existing arterial bypass (epifascial)

Relative contraindications that require a particular balancing of benefits and risks:

- Allergy against compression material
- Inflammatory processes (compression products to reduce inflammation, pain or edema in leg erysipelas or cellulitis should only be used in combination with antibacterial treatment)
- PAOD with ABPI between 0.6 and 0.9 and/or ankle pressure between 60 and 90 mmHg
- Skin diseases with massive exudation
- Primary chronic polyarthritis
- Severe sensory disturbances of the extremity
- Advanced peripheral neuropathy (e.g. in diabetes mellitus)

Risks and side effects

Inelastic wraps can cause skin necrosis and pressure damage to the peripheral nerves if handled improperly. Itching, scaling and signs of inflammation may occur under the compression aids in cases of sensitive skin, so adequate skin care is advised. Regular medical check-ups are required to maintain successful therapy. If acute pain or skin irritation occurs while wearing, immediately consult your physician. In this case, the compression aid should no longer be worn until otherwise prescribed by a physician.

Please note: Consult your physician. The precautions and contraindications listed here are guidelines only and are no substitute for medical advice.

Special instructions

- Ensure the liner remains smooth and free of creases during application.
- Protect the compression product from ointments, oils and greases to preserve its service life.
- Compression products must not be worn over open wounds. Wounds should be covered with appropriate dressings.
- Failure to comply with this instruction may reduce the safety and effectiveness of the product.
- All serious incidents that occur when using the product should be reported to the manufacturer and the competent supervisory authority.
- It should be ensured that the pressure of the compression product decreases (degressive pressure gradient) from distal to proximal (away from the body to close to the body).
- Make sure that the liner does not crease.

Application

Applying the Compressiflex® Standard Thigh

1. Pull on the provided liner. Please make sure that you first follow the instructions for applying the calf and knee compression aid before finally applying the thigh compression aid.
2. Before applying, unfold each hook fastener on the product to its full length, roll the ends back and secure them there so that they cannot get tangled. Slide the foot into the opening between the elastic insert and the product and pull the product up to the thigh, but no further than the liner. The textile label is located on the top at the back of the leg and faces outward.
3. Carefully position the two lowest hook fasteners over the leg and fasten them without applying compression. The hook fasteners should overlap as little as possible without leaving any gap between them. Press the ends of the fasteners firmly to the sides. Continue positioning the remaining hook fasteners in the same way and fix them in place.
4. Apply the Accutabs: your prescribing physician should tell you which compression range is best for you. Apply the Accutabs accordingly and then close the hook

fasteners as directed (refer to the Accutab® instructions for use).

5. Then place the waist attachment around the waist. The belt may be trimmed to desired length. Fasten the wide end of the waist attachment on the belt. Pull the narrow end towards the knee and fasten it on the wrap.

Removing the Compressiflex® Standard Thigh

First remove the belt and the waist attachment. Then loosen each hook fastener, roll their ends backwards and secure them there so that they cannot get tangled. Remove the product from the leg and take off the liner.

Compressiflex® Contain Thigh

Note: If pairing this garment with other Compressiflex Contain products, first don the liners for the thigh and calf (if applicable), secondly the Calf wrap (if applicable), thirdly the Knee wrap (if applicable), and finally the Thigh wrap, following the included instructions for each product.

1. Disconnect each strap and roll the strap backward into a loop, connecting the hook tab to its own strap. Next, with the stretch panel in front, slide the garment over the foot and pull up to the thigh.
2. Beginning with the lowest strap and working up the leg, pull each strap across the leg with comfortable tension, minimal strap overlap, and no gaps between straps, adhering each hook tab firmly onto the opposite side. If desired, the garment's length can be trimmed in the groin or gluteal fold area for comfort.
3. Next, position the belt by wrapping around the waist. Belt may be trimmed to desired length. Attach the wide end of the Hip Attachment to the belt, and pull the narrow end down towards the knee, attaching it to the Thigh garment. If desired, the Hip Attachment can be trimmed in length by removing the hook tab, trimming length, and then replacing hook tab, overlapping the edge by 1". Also, the Hip Attachment can be trimmed in width to create two separate panels by cutting at the center between the two sew lines.

Removing the Compressiflex® Contain thigh

First remove the belt and the waist attachment. Then loosen each hook fastener, roll their ends backwards and secure them there so that they cannot get tangled. Remove the product from the leg and take off the liner.

Storage, wearing period and service life

Unless otherwise prescribed by your physician, the compression product can be worn during the day and overnight. However, remove the product once a day to clean the skin and care for any wounds. The product should then be reapplied according to the instructions for use. How often the product needs to be removed for skin checks, hygiene and/or wound care may vary according to the individual needs of patients and the prescriber's recommendations. A liner should be worn under every compression product. Wearing and washing a compression product every day may cause the medical pressure gradient and elasticity of your wrap to diminish. The

shelf life date is indicated on the packaging using the symbol of an hourglass. We guarantee maximum effect for the subsequent six months of wearing.

Care and storage

Follow the instructions for care on the packaging and/or the textile label. Close all hook or other fasteners before washing. Wash the product in a mesh laundry bag on a gentle cycle using a mild detergent. Do not use bleach or fabric softener. Dry the product on low/no heat in a dryer or air dry (air drying is recommended to extend wrap life). Store the product at room temperature in a dry place away from light. Keep away from direct heat. Wash liners before first use.

Disposal

There are no special requirements for disposing of the medical compression product. It can be disposed of as residual or household waste. Please comply with local regulations for the disposal of textiles.

List of materials

Compreflex® Standard Thigh

Cotton
Polyamide
Polyurethane foam
Neoprene
Elastane
Made without natural rubber latex

Compreflex® Contain Thigh

Polyamide
Polyurethane Foam
Elastane
Made without natural rubber latex

Pack contents

Compreflex® Standard Thigh

1 Compreflex® Standard Thigh
1 waist attachment
1 pair Cotton Liners
1 set Accutab®

Compreflex® Contain Thigh

1 Compreflex® Contain Thigh
1 waist attachment
1 pair Cotton liners

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SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG,
© 2025 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland
SIGVARIS GmbH, Dr.-Karl-Lenz-Strasse 35, 87700 Memmingen, **Germany**
SIGVARIS S.r.l., Via Galileo Galilei 2 / 2, 31057 Silea (TV), **Italy**
SIGVARIS S.A., ul. Mazowiecka 11 lok 49, 00-052 Warszawa, **Poland**
SIGVARIS (Australia) Pty. Ltd., 8/53 Stanley Street, West Melbourne VIC 3003, **Australia**