




Care and Compress

The Venous Leg (1)

What do their legs look like?



Any lower leg oedema present?

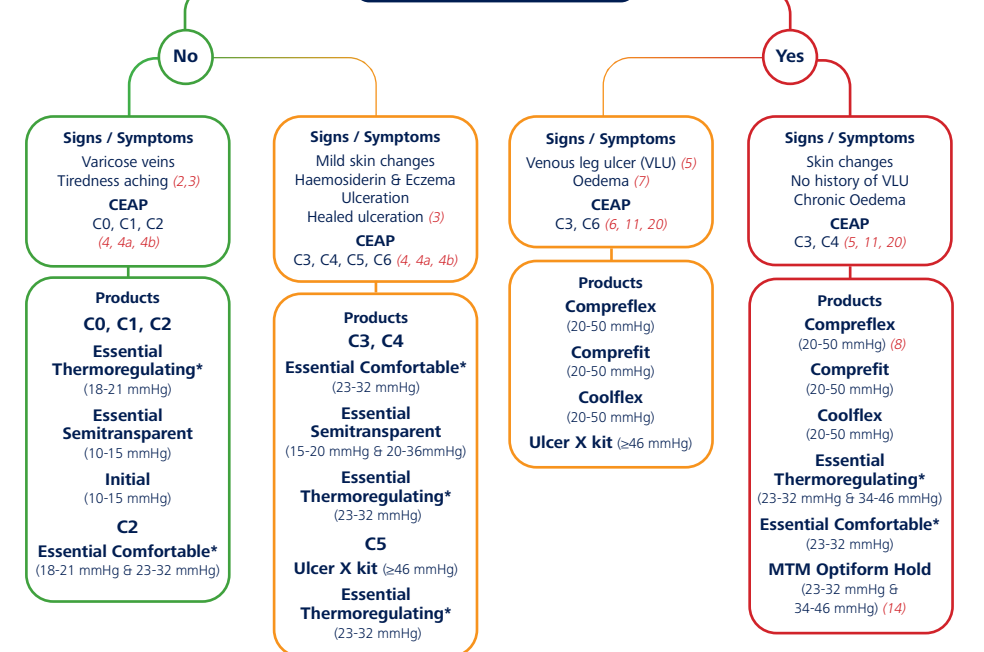


Introduction

SIGVARIS has more than 60 years' experience of producing medical compression garments as a solution for the treatment of venous disease and oedema. Today, millions of legs have worn SIGVARIS medical compression worldwide achieving great results.

As part of our Care and Compress initiative, SIGVARIS are committed to helping improve patient outcomes with compression therapy as a mainstay in the treatment of both venous and lymphatic disease. This would coincide with the work that NHS England are undertaking, looking at chronic wounds as part of 'Leading Change, Adding Value' initiative and in line with the Legs Matter campaign (www.legsmatter.org).

This aid is designed with easy to use flowcharts to help with the decision making process regarding the most appropriate compression garment to use for the condition that is being treated.


* This product is also available in Made to Measure

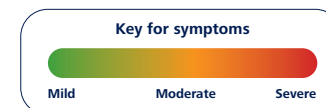


References relating to the condition and management are outlined in *Italic* in brackets. Pictures are used as an example of the condition.

All patients must have had a holistic assessment including ABPI before prescribing garments / medical devices.

Chronic Oedema / Lymphoedema (1,3)

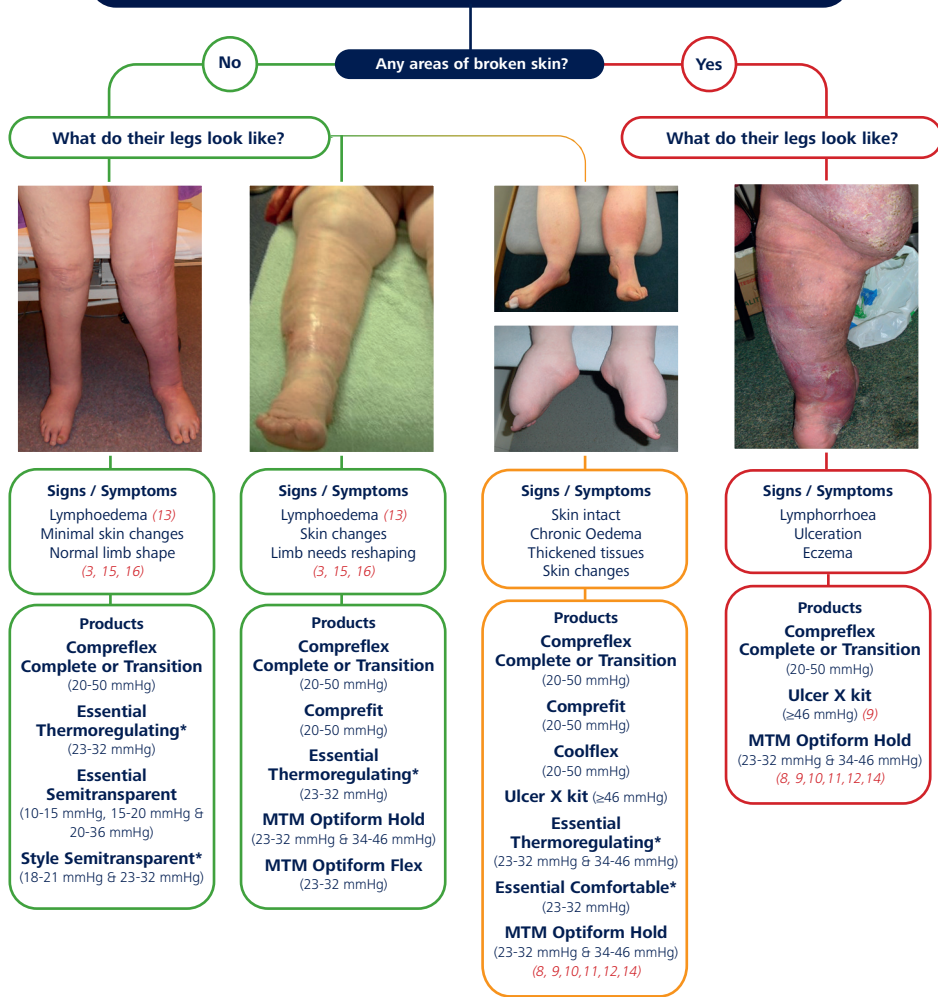


* This product is also available in Made to Measure

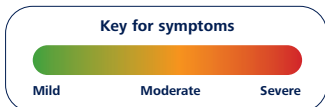


References relating to the condition and management are outlined in *Italic* in brackets. Pictures are used as an example of the condition.

All patients must have had a holistic assessment including ABPI before prescribing garments / medical devices.

Lipoedema / Lipolymphoedema (1, 17)

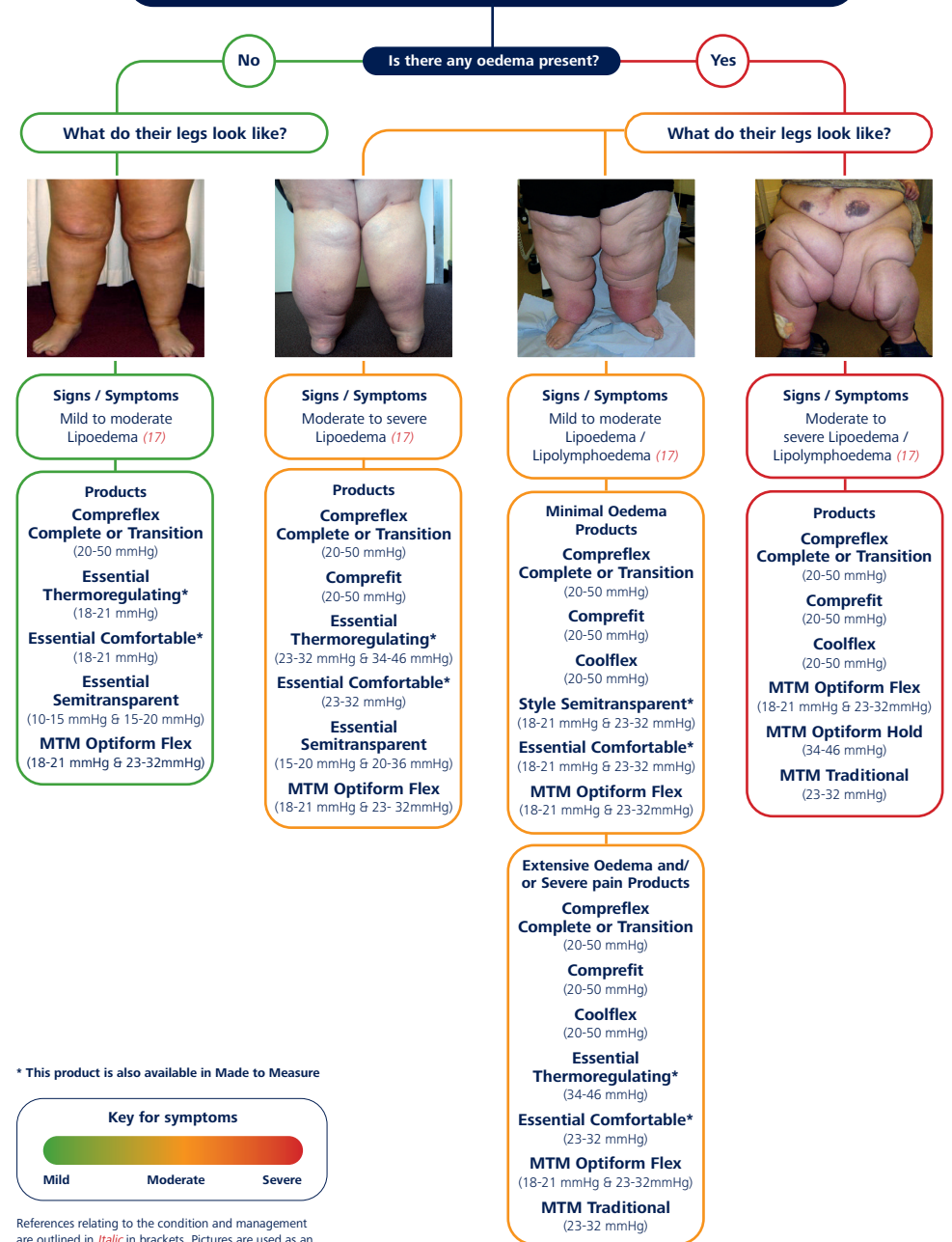

* This product is also available in Made to Measure

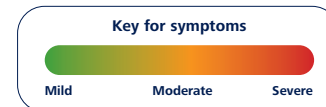

References relating to the condition and management are outlined in *Italic* in brackets. Pictures are used as an example of the condition.

All patients must have had a holistic assessment including ABPI before prescribing garments / medical devices.

Reference

1. Wounds UK (2015) **Best Practice Statement: Compression Hosiery** (2nd Edition), London, Wounds UK
2. Blazek C, Amsler F, Blaettler W et al (2013) Compression Hosiery for occupational leg symptoms and leg volume: a randomized cross over trial in a cohort of hairdressers. **Phlebology**. 28(5): 239-247
3. Rabe E, Partsch H, Hafner J et al (2017) Indications for medical compression stockings in venous and lymphatic disorders: An evidence-based consensus statement. **Phlebology**. Published on-line February <https://doi.org/10.1177/0268355516689631>
4. Porter JM, Moneta GL, International Consensus Committee on Chronic Venous Disease. Reporting standards in venous disease: an update. **J Vasc Surg** 1995; 21:635-45.
- 4a. Lurie F; Passman M et al (2020) CEAP classification system and reporting standard, revision 2020. *Journal of Vascular Surgery: Venous and Lymphatic Disorders*. Vol 8: 3: 342-352
- 4b. www.nice.org.uk/guidance/CG168
5. Wounds UK (2016) **Best Practice Statement: Holistic management of venous leg ulceration**. London. Wounds UK.
6. Davies C, Green T (2017). Treatment of a recalcitrant chronic venous leg ulcer using Compreflex adjustable compression device: case study. **SIGVARIS** Britain Ltd.
7. Mosti G, Picerni P and Partch H (2012) Compression stockings with moderate pressure can reduce chronic leg oedema. **Phlebology**. 27: 289-296
8. Atkins L; Barker C; Green T (2016). Lower Limb Oedema: Case series showing oedema reduction using comreflex adjustable compression system. **SIGVARIS** Britain Ltd.
9. Patullo L et al (2017) Successful outpatient management of lymphoedema and lymphorrhoea with wrap around compression: a case study. **Journal of Wound Care**: Vol 26: No 3: 100-106
10. Williams A (2016) A review of the evidence for adjustable compression wrap devices. **Journal of Wound Care**. 25(5): 242-247
11. Mosti G, Cavezzi A, Partsch H et al (2015) Adjustable Velcro Compression Devices are More Effective than Inelastic Bandages in Reducing Venous Oedema in the initial treatment phase: A randomized controlled trial. **Eur J Vasc Endovasc Surg**. 50: 368-374
12. Ehmann S, Whitaker JC, Hampton S and Collarte A (2016) Multinational, pilot audit of a Velcro adjustable compression wrap system for venous and lymphatic conditions. **Journal of Wound Care**. 25(9): 513-520
13. Damstra R, Partsch H (2013) Prospective, randomized, controlled trial comparing the effectiveness of adjustable compression Velcro wraps versus inelastic multicomponent compression bandage in the initial treatment of leg lymphedema. **Journal of Vascular Surgery: venous and Lymphatic Disorders**. 1(1): 13-19
14. Green T (2016). Optiform made - to - measure flat knit garments in the management of lymphoedema / chronic oedema. **SIGVARIS** Britain Ltd.
- 14a. Green T (2016). Using Optiform flex made - to - measure flat knit garments in managing lower limb lymphoedema. **SIGVARIS** Britain Ltd.
15. Lymphoedema Framework (2006) **Template for Practice: compression hosiery in lymphoedema**. London: MEP Ltd
16. Lymphoedema Framework (2006) **Best Practice for the Management of Lymphoedema. International consensus**. London: MEP Ltd
17. Wounds UK (2017) **Best Practice Guidelines: The Management of Lipoedema London**. Wounds UK. Available to download from: www.wounds-uk.com
18. International Society of Lymphology (2016) The diagnosis and treatment of peripheral lymphoedema: 2016 Consensus document of the International Society of Lymphology. **Lymphology**: 49: 170-184
19. Wound Care People (2019) **Best Practice in the Community. Chronic Oedema**. Wound Care People, Wixford. Available to download from: www.jcn.co.uk
20. Wound UK (2019) **Best Practice Statement: Addressing complexities in the Management of Venous Leg Ulcers**. London: Wounds UK. Available to download from: www.wounds-uk.com

AUTHORS: Tracy Green (Clinical Manager), Susan Knight (Tissue Viability Nurse Consultant)

SIGVARIS Britain Ltd

1 Imperial Court, Magellan Close,
Andover, Hampshire, SP10 5NT

01264 326666

www.sigvaris.group

With more than 150 years
of expertise within medical
compression wear, **SIGVARIS
GROUP** is dedicated to help
people feel their best. Every day

**SIGVARIS
GROUP**