

SIGVARIS

MODE D'EMPLOI

GEBRAUCH

INSTRUCTION

GEBRAUCHSANW

ISTRUZIONE

INSTRUCTIONS FOR USE

MODE D'EMPLOI

GEBRAUCHSANWEISUN

INSTRUCTION

ISTRUZIONE D

MODE D'EMPLOI

GEBRAUCHSAN

ISTRUZIONE I

DOFF N'DONNER

Instructions for use



DOFF N'DONNER SET

DOFF N'DONNER SET, consisting of:

A: DND

B: CONE

DND sets make it easy to roll SIGVARIS medical compression stockings on and off.

Suitable for the following models:

A-D Calf stocking, open / closed toe
Soft top or grip top

A-G Thigh stocking, open / closed toe
Soft top or grip top





A



DOFF N'DONNER

CAUTION: DO NOT HEAT OR PLACE IN DIRECT SUNLIGHT OR ON HEATED SURFACES! Heat will damage this product. Exercise **CAUTION** when using pointed or sharp objects. The product is **NOT** toxic and is filled with soap and water.

CAUTION: If used frequently for a long period of time, the DND may become brittle and may wear out in exceptional cases. We therefore recommend storing the DND in a safe place. We therefore recommend that the DND be stored protected from light and heat.

DND and CONE have been tested only together with SIGVARIS stocking models. SIGVARIS excludes any liability to the extent allowed by law. In particular, SIGVARIS accepts no liability for improper use of the DND and CONE, for user errors or for other harm suffered by the customer/patient as a result of its use, especially in conjunction with products not manufactured by SIGVARIS.



CLEANING THE DOFF N'DONNER

After use we recommend that the DOFF N'DONNER (DND) be cleaned and/or disinfected. For private use it is recommended that the DND be cleaned in a commercially available plastic bag filled with soapy water. For institutional use (e.g. hospitals, doctor's surgeries, medical suppliers etc.), the DND must be disinfected or sterilized.



1

Place the SIGVARIS sterilization bag over the CONE.



2

Pull the DND over the CONE and roll it downward.



Take the lower ends of the sterilization bag and roll these upward over the DND.

4

Fill the sterilization bag with a disinfectant (DND should be lightly soaked in the liquid).

5

Move the sterilization bag up and down so that the disinfectant is distributed well and evenly through the DND.



6

Roll the sterilization bag off with the DND.

7

Empty out the liquid.

8

Take the DND out of the sterilization bag and allow it to dry thoroughly.

9

Dispose of the sterilization bag (for one-time use only).

INSTRUCTIONS FOR USE

DOFF N'DONNER INSTRUCTIONS FOR USE

The DOFF N'DONNER (DND) is a very soft device that allows you to quickly and painlessly convey medical compression stockings over an arm or a leg. The DND is suitable both for self-application and assisted application. The CONE simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND. By virtue of the integrated suction cup, the CONE can be attached well to smooth surfaces.

INSTRUCTIONS FOR USE



Clamp the CONE to a smooth surface.
(Press the lever down, CONE adheres firmly.)



Place the compression stocking with the finished top edge (soft top or grip top) over the CONE. The inside of the stocking rests on the CONE.



Roll the compression stocking completely over the CONE.

Closed toe: Place seam stretched smoothly onto the CONE TIP.

Open toe: Place at a width of 3 fingers from the top.



Place the DND over the CONE and roll it downward. Pull the finished edge of the compression stocking up over the DND.



Calf stocking: Roll the DND upward until the compression stocking is rolled completely onto the DND.

INSTRUCTIONS FOR USE



Thigh stocking: Pull the finished edge of the compression stocking beyond the tip of the CONE, so that the stocking is wrinkle-free. Grasp the DND and pull it upward until the compression stocking is rolled completely onto the DND.



Stretch the toe seam, so that it is clearly visible from the outside. Hold the DND in such a way that the heel of the compression stocking is visible from the outside and is facing the floor.

INSTRUCTIONS FOR USE



Place the toe seam on the toes. The heel of the rolled-up compression stocking points downward.

Open toe: Place the finished cuff as shown by the arrow.



Roll the DND over the heel up onto the leg.



As soon as the compression stocking fits on the leg, roll the DND down over the heel once again.



10

Spread the compression stocking over the leg with special gloves for a wrinkle-free fit.



To watch video instructions,
please visit our website:
www.sigvaris.com/videos

Composition:

- DND:** Thermoplastic elastomer
Contents: 99.8% soapy water, 0.02% polyacrylamide
- Cone:** Polyethylene

SIGVARIS AG, Gröblistrasse 8, 9014 St.Gallen, Switzerland
Phone +41 71 272 40 00, certified ISO 9001, EN ISO 13485

SIGVARIS GmbH, Dr.-Karl-Lenz-Strasse 35, 87700 Memmingen, Germany
Tel. +49 8331 757 0

SIGVARIS medizinische Kompressionsstrümpfe m.b.H., Hietzinger Hauptstrasse 22/A/2/5, 130 Wien, Austria
Tel. +43 1 877 69 12

SIGVARIS S.r.l., Via Galileo Galilei 2/2, 31057 Silea (TV), Italy
Tel. +39 800 178486

SIGVARIS (Australia) Pty. Ltd., 207 Victoria Street, West Melbourne VIC 3003, Australia
Phone 130 SIGVARIS (1300 744 827)

If you look for further distributors please consult: www.sigvaris.com

